

### PARTS OF YOUR BladeFLASK:

**INDICATOR:**  
SHOWS WHICH WAY SHARP  
EDGE MUST FACE.

THIS PART OF  
THE SCALPEL  
HANDLE IS  
CALLED THE  
TANG.

COUNTER

LANDING  
PAD




MOUNTING  
BRACKET

### MOUNT TO WALL OR TROLLEY:

1. If you haven't mounted your BladeFLASK before, prepare mounting bracket\*.

**A** BRACKET HAS DOUBLE SIDED  
TAPE ON BOTTOM

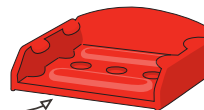


**B** OR USE SCREWS.



\* See fitting template and instructions inside  
the bracket packaging.


2. BladeFLASK can  
be mounted on  
a wall or on a  
trolley or bench.

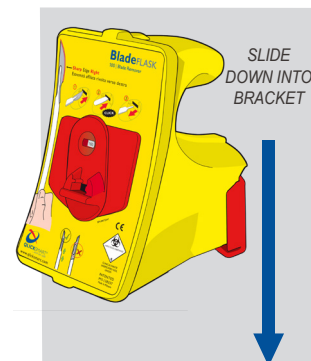


**NOTE:** BladeFLASK is  
mounted on Mounting  
Bracket during use.

### REMOVE BLADE SINGLE-HANDED ACTION:


1. Place blade on  
landing pad with  
the sharp edge  
facing to the  
right (or 'tang'  
up).

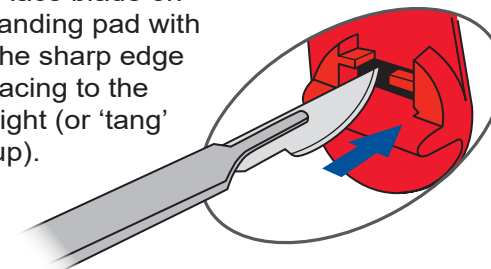

2. Push scalpel  
handle into  
BladeFLASK  
until you  
hear a click.

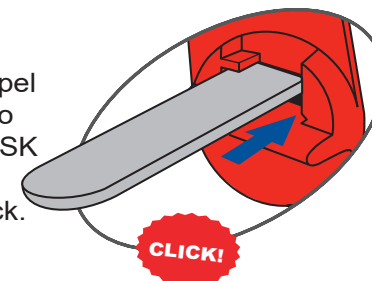

3. Remove handle.

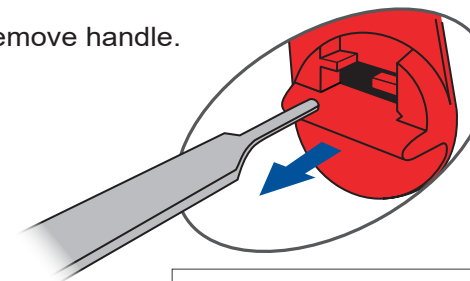

**NOTE:** BladeFLASK is disposed  
of as medical waste after 100  
blades have been inserted.