



Top Tips: Using the **BladeFLASK**[®] scalpel blade remover

- 1. Single-handed:** Have Mounting Bracket installed for single-handed blade removal.

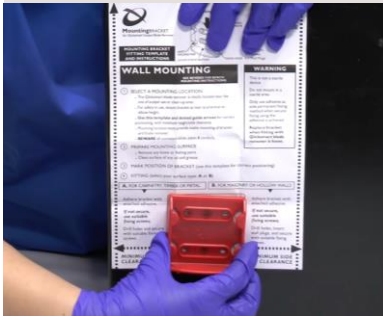

Tip: Installation guide is included in Mounting Bracket packaging.

- 2. Blade direction:** Blade must face same direction as symbol on label.

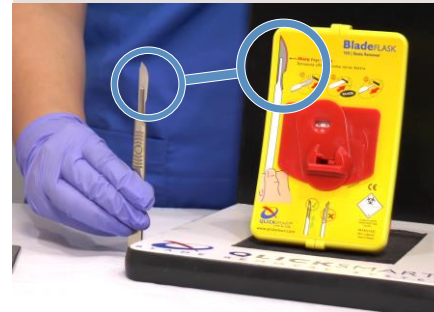

- 3. Remove blade:** Push scalpel handle into BladeFLASK until the “click”.



Tip: The “click” indicates safe removal and containment of blade.

- 4. Replace?:** Check the automatic counter to see when replacement is needed.



Tip: The automatic shut-off activates after ~100 blades are removed.



Scan the QR Code to see **BladeFLASK** in action. Contact us to arrange sharps safety education for your facility:
w | www.qlicksmart.com e | hello@qlicksmart.com p | +61 7 3844 1182